



1. Open up the box from the top.

2. Once opened, notice the cardboard packing.





3. Pull out cardboard cover.

4. Once the cardboard cover is pulled out, notice the two pockets. These can also be removed.






5. Pull back the cardboard lever covering the head portion. This flap is removable as well if needed.

6. With assistance gently take out the unit as so or vise versa if you are packing the unit back into the box.



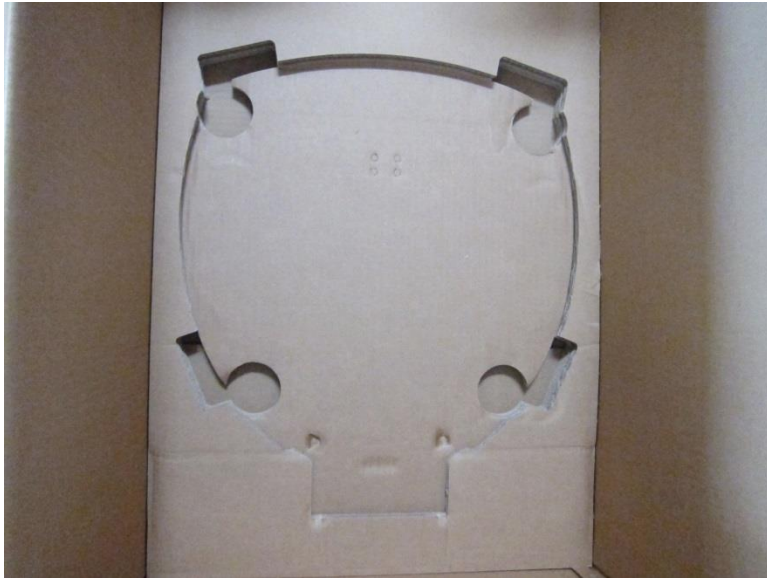

7. Custom cutout designed to specifically fit the foot supporter pegs.

8. This is how the box will look once its been emptied out. Please make sure to properly pack your unit if doing so by inserting the InBody into the cutouts. When repacking, inserting the cardboard pockets in vise versa order will suffice.

